
## INSTRUCTIONS FOR USE

### COMPATIBILITY

The adapter accepts any standard ECG lead connector with a 3mm–4mm banana plug. The use of electrodes within the range of dimensions specified in the illustration at right is recommended.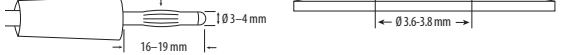

### SAFETY INFORMATION

In these User Instructions, the word **WARNING** indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

<div><span><span> </span></span></div> <b>WARNING</b>
Do not use with MRI equipment.

### CLEANING AND CARE

- Remove the banana plug and open the latch prior to cleaning.
  - Dampen a clean cloth or gauze pad with one of the following appropriate cleaning solutions and wipe all exposed surfaces:
    - green soap - green soap tincture
    - alcohol free hand soap - 2% glutaraldehyde solution
    - sodium hypochlorite (bleach) solution 10% in water
  - Dampen a clean cloth or gauze pad with sterile or distilled water and again wipe all exposed surfaces.
  - Dry all exposed surfaces with a clean, dry cloth or gauze pad.
- NOTE:** May be subjected to 10 sterilization cycles by ethylene oxide method.

<div><span><span> </span></span></div> <b>CAUTION</b>
---

- Inspect the Sure-Lock clip prior to use. Do not use the Sure-Lock clip if there are any signs of internal contamination or obstructions.
- Cleaning with alternate compounds and methods may damage product. Contact Curbel for cleaning assistance.
- Never immerse in any liquid. Doing so may cause damage to the clips, including corrosion.
- Do not continue to use if you see corrosion of the metal spring or deterioration of the plastic housing is evident.
- Do not sterilize by autoclave, radiation, or steam.

For professional users in the European Union, if you wish to discard electrical and electronic equipment (EEE), please contact your dealer or supplier for further information. Notice to user and/or patient that if any serious incidents have occurred with the use of the device, the incident should be reported to the manufacturer, as well as the local competent authority of the member state.

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